1000 Series Owner's Manual

: 1-800-265-9830

: 1-877-636-8944

: info@advancedhealthcare.ca

 2221 46th Avenue, Lachine, Quebec H8T 3C9, Canada

Table of Contents

1. Technical Specifications	2
1.1 Weight Capacity	2
1.2 Bed Weight	2
1.3 Features & Optional Accessories	2
1.4 Storage	2
1.5 Moving Specifications	2
1.6 Measurements	2
1.7 Limited Warranty	3
1.8 Return Policy	3
2. Unpacking Directions	4
3. Parts List	6
4. Assembly Instructions	7
5. User Instructions	11
5.1 Warnings	11
5.2 Inspection before use	12
5.3 Function & operation	13
5.3.1 Remote control	13
5.4 Maintenance	13

1. TECHNICAL SPECIFICATIONS

1.1. **Bed weight capacity**

The maximum weight capacity for this bed is 170 kilos (375 lbs).

1.2 **Bed weight**

The bed weights ~ 94 kilos (~206 lbs).

The weight of the bed may vary depending on the options and accessories installed on it. For easy access, transport, and storage, the bed separates in 4 parts. Refer to section 2, on p.9 for instructions on how to assemble and dismantle the bed.

1.3 **Accessories**

Features:

- Remote control
- Emergency battery (for temporary electronic functions during power outage)
- Casters with individual braking system
- Head board
- Foot board

Optional:

- Half side rails
- Full length side rails
- Bar with trapeze
- Bed trolley

1.4 **Storage**

The bed must be kept in a dry and well ventilated area.

1.5 **Moving specifications**

Before moving the bed, it must be at its lowest position. Always use the head board or the foot board of the bed to move it. Never use the side rails for moving the bed.

Apply the brakes when the bed is in place.

1.6 **Measurements:**

1.6.1 Maximum angles

- Maximum backrest: ~75 degrees
- Maximum footrest: ~40 degrees
- Trendelenburg: ~12 degrees

1.6.2 Overall Size: 220 x 103 cm

1.6.3 Platform Size: 196 x 90 cm

1.6.4 Minimum height: 28 cm

1.6.5 Maximum height: 75 cm

Note: The dimensions specified do not take the manufacturing tolerances into consideration.

1.7 Limited Warranty

Advanced Health Care Products distributes high quality beds. Our products are guaranteed against manufacturing defects up to one year after the bed is purchased. Furthermore, we offer a specific 10 year guarantee on the frame and 2 years on the electric motors, hand control, and electronic components following the date of purchase.

Advanced Health Care Products reserves the right to substitute equal or superior quality materials during repairs and/or replacements. All repaired or replaced parts are guaranteed for whichever is greater; 1 year from the date of service or the life of the warranty.

This warranty does not cover damages caused by negligence or inappropriate use. Advanced Health Care Products will not be held responsible for any other warranty offered by any person, firm or company, to the exclusion of the one stipulated above.

This warranty gives you specific legal use rights and is only valid for the original purchaser or for anyone who would have received it as a gift.

For further information, contact one of our customer service representatives at 1-800-265-9830.

1.8 Return policy

1.8.1 Non compliant product

If an Advanced Health Care Products product is not according to the original order invoice, the problem must be notified within 48 hours after the time of delivery. Actions necessary to correct the situation will be at Advanced Health Care Products expense and will be made as soon as possible, following the notification of the non compliancy.

1.8.2 Damaged product

It is the responsibility of the receiver to verify the shipment even if the box (es) seems to be in good condition. If there is any damage, it must be indicated on the bill of lading. Advanced Health Care Products must be informed of the situation within 24 hours of receipt of goods in order to notify the carrier as soon as possible. If Advanced Healthcare Products is not informed within 24 hours or the damages are not indicated on the bill of lading, the customer may have to pay all the costs incurred to replace or repair the damaged product.

1.8.3 Returned product

Returned goods require a RA number and are subject to a 25% restocking fee. Furthermore, the customer must pay the transport charges. For more information, please contact our customer service representatives at 1-800-265-9830.

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

Step 3: Carefully remove half of the bed deck at a time while paying attention not to scratch the headboard or footboard. Unscrew with allen key, unlock wheels and carefully remove the bed from the wood crate.

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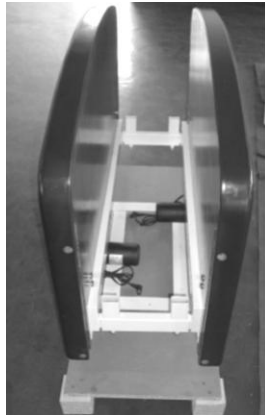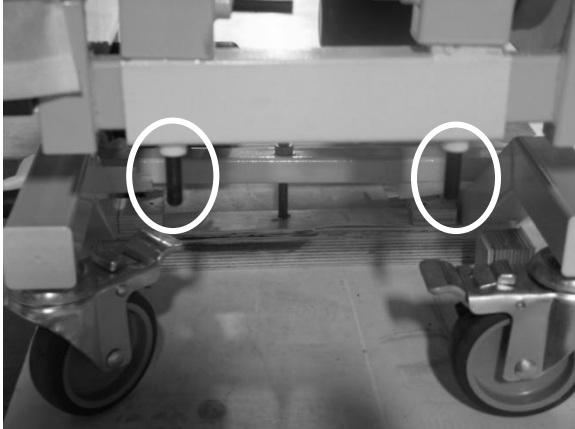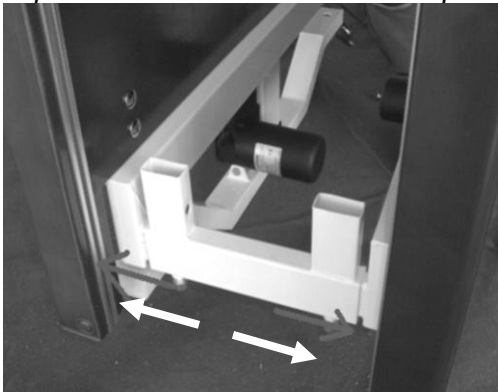
4. ASSEMBLY INSTRUCTIONS *INSTRUCTIONS D'ASSEMBLAGE*

Lay side panel (B) flat on the ground and insert bed frame (A) into reception tube of side panel (B), Fig.1.

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Fig.1

2. Repeat above instruction for the second half of the bed frame.

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10.1. Lift selected bed frame to an upright position, Fig.9.

Remonter la tete du lit, Fig.9.

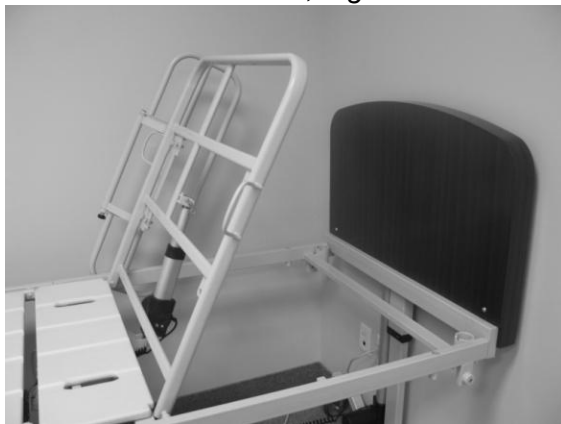
Fig.9

10.2. Align holes on side rail mounting plate with the corresponding holes on the bed frame, Fig.10.

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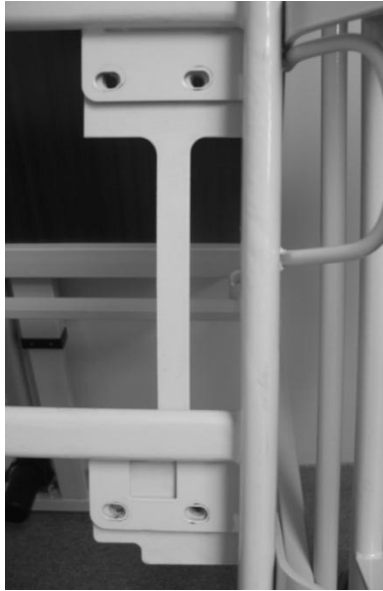
Fig.10

10.3. Use 4 bolt screws (K) and 4 metal washers (L) to secure the rail to the bed frame.

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11.1. Place side rail mounting screw (I) through the mounting hole on each side of rail and place plastic washer (J) between rail and the bed.

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11.2. Align rails with receiving holes on bed, Fig.11.

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5. USER INSTRUCTIONS

5.1 Warnings

Please read the instructions in this manual carefully before using this bed:

- This bed is not for pediatric use;
- This bed should not be used in the presence of flammable anesthetic gas mixed with air, oxygen, or nitrogen oxide;
- Always apply the brakes after moving the bed and before a person sits or lies down in the bed. This procedure is important in order to reduce the risk of injuries when the user is getting in or out of the bed. Try to move the bed to make sure that the brakes are applied;
- Take into consideration the general condition of the user before automatically raising the half side rails at the foot end of the bed, so that the user can get out of the bed at their convenience;
- Half side rails at the head of the bed should always be up when operating;
- Unless otherwise medically advised by a professional, leave side rails up and locked when the user is sleeping or left without supervision;
- When the side rails are up, try to move them to make sure that they are properly locked; Side rails are made to prevent accidental falls. They must not be used to restrain, to keep the user from getting out of the bed, or to help the user to turn. It is the caretaker's responsibility to use the appropriate restraining means, for the user's safety.
- Always lower the bed to the lowest position or leave side rails up when the user is sleeping or left without supervision to reduce risks of falls or injuries;
- Always make sure nothing interferes with the functioning of the bed before using the commands, i.e.: user or staff's limbs, medical equipment, etc..... which could result in injuries and/or equipment failure;
- Make sure the power cable is properly fixed so it cannot get damaged during the functioning of the bed and to avoid injuries and/or equipment failure;

- Unless otherwise proscribed by a medical professional, always place the platform horizontally when the user is sleeping;
- If required for the user's welfare or security, unplug the remote;
- To reduce the risk of injuries when moving the bed with a person laying in it, always put the bed at its lowest position, then place the platform horizontally and leave the side rails raised;
- Unplug the bed immediately if a significant overflow of liquid is accidentally spilled on the electronic components, the wires, or the electric motors. The functioning of these components can be affected. The user must be removed, the bed cleaned and examined by a qualified technician. The bed can be used again only if it has been properly cleaned and examined;
- To clean the bed, use detergents normally used in hospitals or at home. The products may be applied by spraying. Always unplug the bed before cleaning. May be steam washed, but do not use high pressure steam. Never use high pressure water, or ultrasounds, and never submerge the bed parts to clean them. Make sure the cleaned parts are dry before using the bed again;
- When changing an electric component, unplug the power cable and the battery fuse. This must be done by a qualified technician;
- A regular maintenance is essential to maintain the bed's reliability. A defective bed should not be used;
- Make sure to block the frame of the bed if you need to work under the platform. This will prevent the bed's structure from falling accidentally;
- The manufacturer reserves the right to modify the bed;
- Only use Advanced Health Care Products replacement parts

5.2 Inspection before use

These verifications are necessary to make sure that the bed was not damaged during the transport:

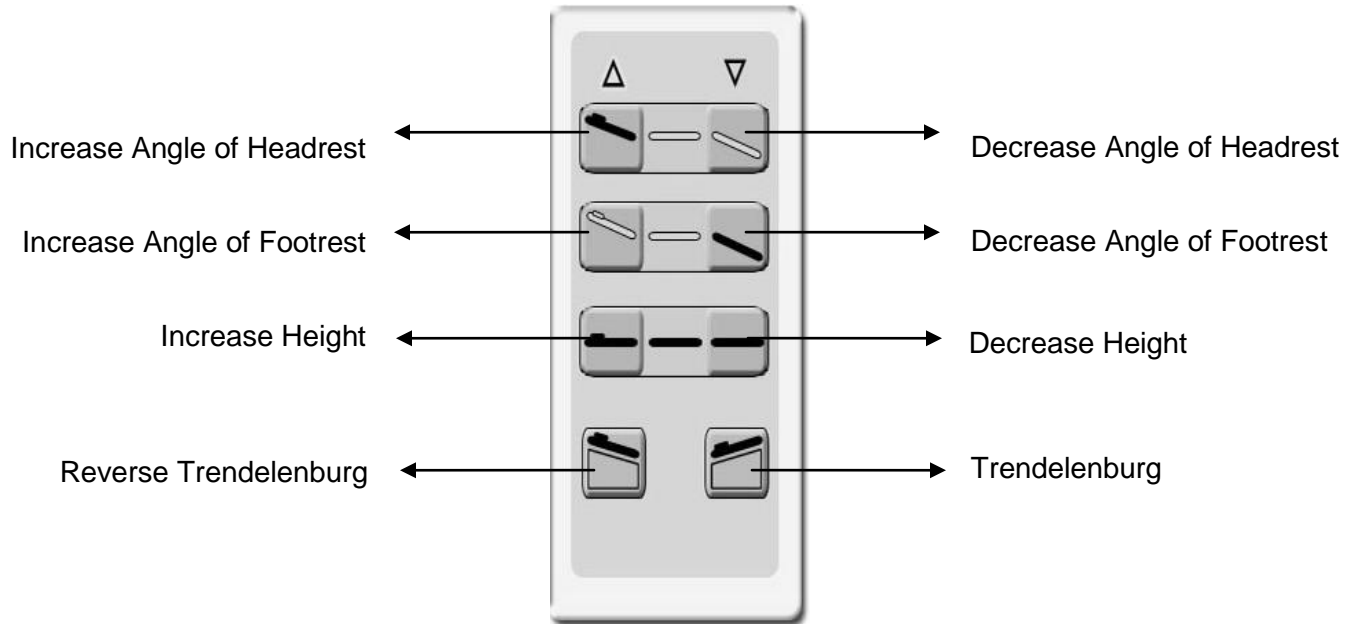
- Inspect the bed and its accessories visually;
- Inspect the head and foot board installation;
- Verify the functioning of the side rails and their locking mechanisms;
- Apply the brakes of the casters and check their functioning;
- Plug the power cable and activate all the motors. Check if their functioning is constant and complete. This verification also allows you to check the functioning of the remote control;
- Unplug the power cable and turn on the motors to check the battery.

Note: Plug the bed for 48 hours consecutively to charge the battery before doing this verification. If the bed is damaged or does not work properly, inform maintenance services or the retailer immediately.

5.3 Function & operation

5.3.1 Remote Control

The standard remote control allows you to use following functions:

5.4 Maintenance

All defects must be repaired immediately by a qualified and authorize technician. The maintenance of bed must be done once a year.