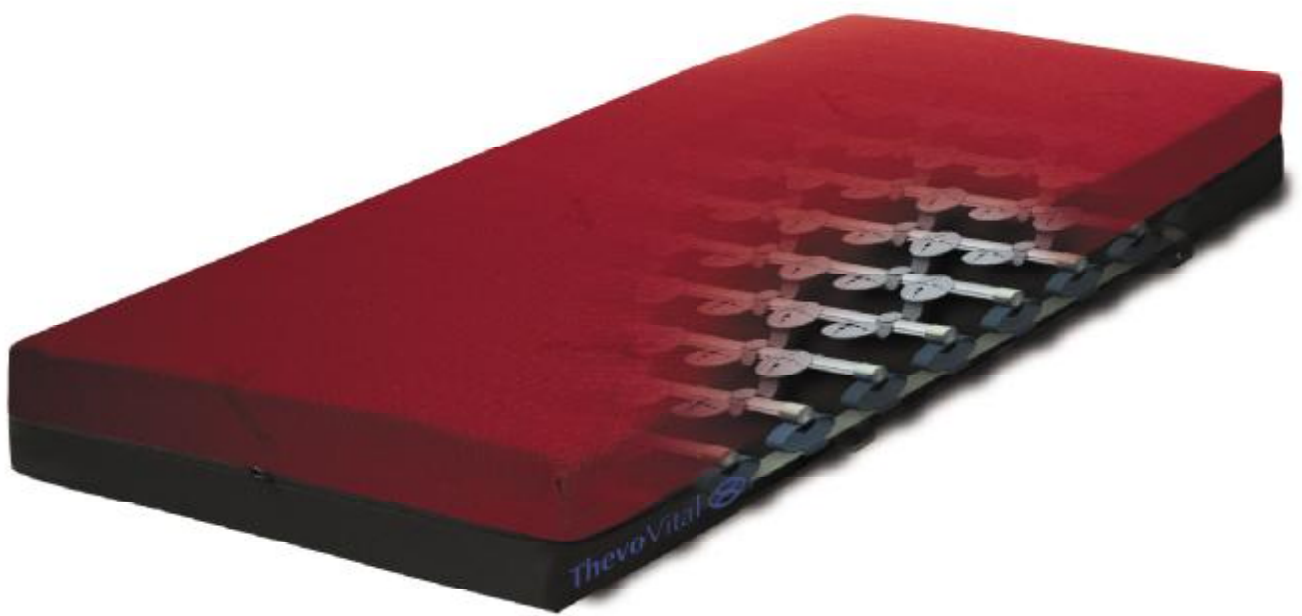
ThevoVital

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Dear User,

You have just acquired a product - the ThevoVital movement bed - that meets the highest quality standards. ThevoVital features the latest materials for an extended service life that's guaranteed. Please read through this manual carefully before using ThevoVital. It contains lots of useful information and tips. If you have any further questions or suggestions concerning ThevoVital give us a call.

Further information is also available from www.thevo-division.com

Our address: Thevo by Thomashilfen
 Walkmühlenstrasse 1
 27432 Bremervörde, Germany
 Tel.: +49 (0)4761 / 886-63 or -68
 www.thevo-division.com

Contents

Page

Technical data	7
Scope of delivery.....	7
Usage.....	7
Disinfection methods and cleaning information.....	7
Application risks and contraindications.....	7
Warranty conditions.....	8

Technical data

ThevoVital external dimensions	90 x 200 cm (35.4 x 78.7 ")	100 x 200 cm (39.4 x 78.7 ")	Height
ThevoVital 80	18.0 kg (39.7 lb)	20.0 kg (44.1 lb)	19.2 cm (7.6 ")
ThevoVital 140	19.5 kg (43.0 lb)	21.5 kg (47.4 lb)	

Other dimensions available on request.

Scope of delivery

- ThevoVital with jersey or incontinence cover
- User manual

Usage

The product is compatible for use with every type of bed. Lay ThevoVital with the red side facing upwards on the existing bed slats or bed base.

Important hint: ThevoVital can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 10 cm (3,9 ") wide.

Attention: Beds with side-rails may require additional side-rail height extenders.

Disinfection methods and cleaning information

Mattress

The washable covers are easy to remove thanks to an integrated zip.

Jersey cover (fabric cover)

- washable at 65 °C (149 °F)
- suitable for tumble dryers

Incontinence cover (rubberised cover)

- washable at 65 °C (149 °F)
- suitable for tumble dryers
- to be cleaned by wiping disinfection

Disinfection

We recommend two disinfection methods accepted by the Robert Koch Institute:

- the vacuum/steam/vacuum method up to a maximum temperature of 110 °C (230 °F)
- the steam flow method up to a temperature of 110 °C (230 °F)

Important hint: A thermal treatment over 110 °C (230 °F) causes a premature ageing and deformation of the plastic parts and should therefore be avoided. Please make sure when using chemical disinfectants that they are suitable for plastic.

Application risks and contraindications

ThevoVital is designed to improve quality of sleep, especially for people with dementia.

The mattress is quite firm. Skin condition should be checked regularly with persons suffering illnesses affecting the skin properties. This particularly applies to immobile patients. The correct mattress density is extremely important in such cases.

The use of ThevoVital does not require any special instructions. Referring to the previous practical use and several tests executed, we are not knowledgeable of any problem in the use of this product.

When used properly, risks can be excluded.

Important hint: For the correct supply it is important to ensure that the mattress is selected according to the body weight of the user in question.

- ThevoVital 80 up to a body weight of 80 kg (176.4 lb)
- ThevoVital 140 up to a body weight of 140 kg (308.6 lb)

ThevoVital 140 may also be used in the border zone of patients weighing 80 kg/176.4 lb (+/- 2 kg / +/- 4.4 lb).

Caution: ThevoVital is not suitable for the prevention and treatment of decubitus ulcers. This requires the use of either ThevoautoActiv or ThevoActiv. We would be pleased to advise you.

Warranty conditions

The company Thevo by Thomashilfen takes full responsibility for the quality of the product supplied. The highest production standards are used in the selection and processing of materials. All ThevoVital mattresses are subjected to a thorough final inspection. Thevo by Thomashilfen offers you a warranty of 24 months. Please retain all relevant ThevoVital receipts. The warranty covers all claims arising from manufacturing or material defects that affect product function. Damage arising from inappropriate use and natural wear and tear is excluded.

Important hint: ThevoVital is autoclavable and may be re-used. ThevoVital should be cleaned and disinfected using one of the cleaning methods stated on page 7 with every change of user. Servicing by the manufacturer is not necessary.

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